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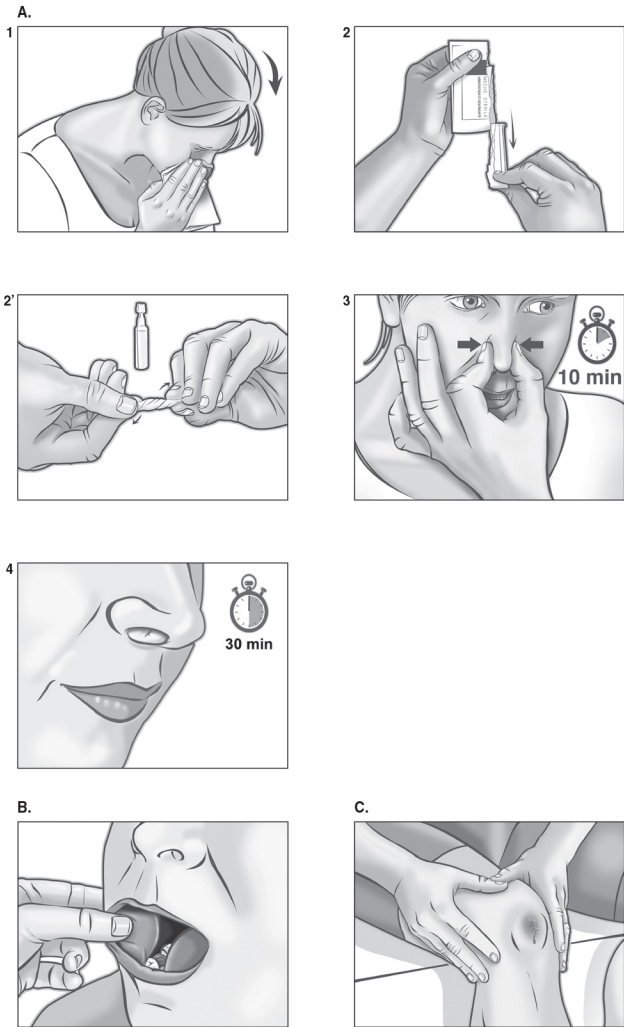
HAEMOSTASIS AND HEALING

1. DESCRIPTION OF THE DEVICE

- a. NAME**
Stop Hém[®]
- b. COMPOSITION**
Calcium alginate fibers (from plants)
- c. FORM AND PRESENTATION**
4 cm swab (5/box)
Individually-wrapped pouches
- d. PROPERTIES**
Stop Hém[®] is a haemostatic and healing swab.

2. INDICATIONS

HAEMOSTASIS	HEALING
• Nosebleeds (Epistaxis) • Topical bleeding (cuts, scratches, puncture sites) • Oral bleeding • Dental extractions	• Topical wounds ≤ 8cm² - Surgical wounds - Traumatic wounds (scratches, cuts, scrapes, open blisters, superficial burns)

Stop Hém[®] is efficient in patients with bleeding disorders, whether congenital or acquired (due to antiplatelet aggregation inhibitor-, Vitamin K antagonist- or heparin treatments). Stop Hém[®] can be used in children, pregnant women and adults.

3. CONTRAINDICATIONS

None known to date.

4. SIDE EFFECTS

Stop Hém[®] contains trace amounts of surfactant and may cause hypersensitivity reactions in some sensitized individuals.

5. PRECAUTIONS

Stop Hém[®] can only be moistened with a saline solution NaCl 0.9%: the Na⁺ ions of the NaCl solution exchange with the Ca²⁺ ions of Stop Hém[®], thereby enabling the gelling and painless removal of Stop Hém[®]. Check the integrity of the pouch before using the product.

6. WARNINGS

Always use Stop Hém[®] in full (without cutting or pulling apart).

Stop Hém[®] should not remain in place for more than 24 hours. Single use. Do not reuse due to the risks of microbial contamination and loss of performance.

- 7. DIRECTIONS FOR USE**
- A - Nosebleed (Epistaxis)**
1. Lean forward and gently blow the nose to clear any clots
 2. Tear open the pouch along the dotted line. Twist Stop Hém[®] to obtain a compact swab. Stop Hém[®] can only be moistened with saline NaCl 0.9% solution for easier insertion into the nostril.
 3. Insert Stop Hém[®] into the nasal cavity and pinch both nostrils for 10 minutes.
 4. Maintain Stop Hém[®] in place for a minimum of 30 minutes and a maximum of 24 hours, use the pouch to discard the swab.
- B - Oral bleeding / Tooth extraction**
(lips, gums, tongue,...)
Apply Stop Hém[®] on the wound, remove when the bleeding stops.
- C - Topical wounds (haemostasis and/or healing)**
Clean the wound with water and soap (or with saline solution NaCl 0.9%).
Apply Stop Hém[®] on the wound and:
- remove once bleeding stops
- or leave in place for healing and remove Stop Hém[®] within 24 hours.
In case of persistent bleeding, seek medical advice.
Fibers potentially remaining after Stop Hém[®] removal shall be eliminated naturally.
Stop Hém[®] may be used with lidocaine-naphazoline.

8. INTERACTIONS

Stop Hém[®] should not be used with alkaline solutions, such as solutions of sodium hypochlorite, or containing chlorhexidine, benzalkonium chloride or copper salts. If such products are used on the wound, rinse thoroughly with a solution of NaCl 0.9% before applying Stop Hém[®].

9. STERILIZATION

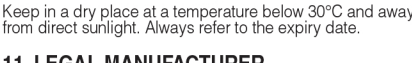
Sterile product, sterilized by β radiation.
Do not re-sterilize.

10. STORAGE

Keep in a dry place at a temperature below 30°C and away from direct sunlight. Always refer to the expiry date.

11. LEGAL MANUFACTURER AND CE CERTIFICATE HOLDER

Les Laboratoires Brothier
41, Rue de Neuilly - 92735 Nanterre Cedex - FRANCE
www.brothier.com - info@brothier.com
Manufactured in France by Les Laboratoires Brothier.

CE-mark identification number of notified body
Batch number
Use by: year and month
Attention: see instructions for use
Haemostatic and healing swab

Distribucja : FUHmann-Eyrard SA (CH), www.stophemo.ch - MediInnovation Sp. z o.o. (PL), www.stophemo.pl - Cimifar Christos Miralis & Co. (GR), www.cimifar-miralis.gr - GSA Healthcare (BE), www.gsa-healthcare.com